PUSH ORTHO ANKLE BRACE AEQUI JUNIOR

Please read this product information carefully before using the Push Aequi Junior.

INTRODUCTION

Push products provide effective solutions for a variety of disorders of the locomotor apparatus. The products can also be used as a preventive measure after previous injuries of the joints.

The Push ortho ankle brace Aequi Junior is a Push Braces product from the so-called ortho category. Push ortho products are innovative and are carefully manufactured. They aim to provide the highest level of comfort, maximum support and effective protection. The Aequi Junior ankle brace was especially designed for children with unstable ankles.

The Aequi Junior is characterised by a high degree of support, effectiveness, comfort and quality. The product provides enhanced stability and helps control pain. A major advantage of wearing this brace is that it increases load bearing capacity. This restores the ankle joint's full functionality more quickly, and most importantly, more safely.

PRODUCT CHARACTERISTICS

The Aequi Junior is an effective ankle brace for children (or adults with a small instep) ages between 6 and 12, with an instep of 23-27 cm (shoe sizes 30-37). UK: instep of 9-10½" (children shoe sizes 12-13 and adult shoe sizes 1-4). This ankle brace is suitable for both the left and right ankle and is available in a single size.

The brace offers a great balance between effective mechanical ankle support and optimal wearer comfort. It consists of a sturdy foam shell and a set of elastic and non-elastic straps. The Aequi Junior provides the required support of the ligaments.

Elastic straps also provide pressure around the ankle joint.

The inversion-eversion movement (inward-outward movement of the foot) is restricted, while plantar-dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running.

The numbers on the straps allow for an easy fitting of the brace.

Tests have shown that children are capable of fitting the brace themselves.

For proper functioning of the brace, however, it is important that parents or carers will observe and provide any help needed and inspect whether the Aequi Junior has been fitted correctly.

Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. When children play indoors, the Aequi Junior can also be worn without a shoe, while retaining the function of stabilisation.

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INDICATIONS

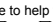
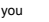
- In the case of early functional therapy for injuries to the medial or lateral ankle ligaments
- Follow-up treatment of ankle fractures that have been operated on or conservatively managed
- In the case of (residual) instability (increased tendency to sprain)
- To prevent (recurrence of) ankle ligament injuries

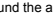
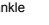
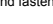
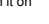
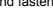
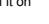
There are no known disorders where the brace cannot or should not be used. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or the symptoms worsen, it is advisable to consult your doctor or therapist.

Always follow the advice of your doctor or therapist.

FITTING

The hook and loop fasteners are marked with symbols. These are to help you fasten the hook and loop material in the correct sequence. First , then .

- The Push Aequi Junior is worn over the sock.
- Place the brace around the foot. Lay the long straps to the sides.
- Fix the diagonal straps  and  and the shell closely around the ankle (Figure A).
- Wrap the long elastic strap  diagonally across the instep and fasten it on the hook and loop material .
- Wrap the long elastic strap  diagonally across the instep and fasten it on the hook and loop material .

If the brace is worn by a child, always check whether it has been fitted properly. If the product needs to be left on overnight, it should be fitted less tightly.

WASHING INSTRUCTIONS

For fine good hygiene and a longer lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push Aequi Junior requires a correct diagnosis of the causes of pain or complaints. Always consult your GP or physiotherapist or a specialist before using the brace.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Check your Push brace before each use for signs of wear or aging of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- To promote the useful life of the brace, it should not be exposed to temperatures above 70 °C.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.
- Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. Be careful not to slip when the brace is worn without a shoe.

ANY QUESTIONS?

For any questions concerning the product, please consult the reseller who provided the brace, or send an e-mail to info@push.eu.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, refer to www.push.eu/distributors

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